

# CORMAY ASO CALIBRATOR

## CALIBRATOR FOR DETERMINATION OF ANTI-STREPTOLYSIN O LEVELS



<b>Kit name</b>	<b>Kit size</b>	<b>Cat. No</b>
CORMAY ASO CALIBRATOR	1 x 5 ml	4-278

### INTRODUCTION

Most people infected with hemolytic streptococcus produce anti-streptolysin O (ASO), antibodies against streptolysin O (SLO), an exotoxin of Streptococcus. Measuring the level of ASO is effective for diagnosing, judging the progress of medical treatment, and assessing recovery from diseases caused by hemolytic streptococcus such as rheumatic fever, acute glomerulonephritis, scarlatina and tonsillitis.

CORMAY ASO CALIBRATOR is prepared by diluting anti-streptolysin O with physiological saline containing 1 w/v% bovine serum albumin to make a 500 U/ml solution.

CORMAY ASO CALIBRATOR should be used for the preparation of the calibration curve for PRESTIGE 24i ASO, CORMAY ASO and CORMAY ASO 500 kits.

### STABILITY AND STORAGE

The calibrator is ready to use. Unopened calibrator is stable at 2-10<sup>0</sup>C until the expiration date printed on the vial. Protect from light.

### WARNINGS AND NOTES

- Product for in vitro diagnostic use only.
- The calibrators should not be mixed with ones in the other vial.
- The calibrator should be used according to this kit insert.
- The calibrator is based on human serum that has been tested for the HIV antibody, HBsAg and HCV and found to be non-reactive. However this material should be handled as though capable of transmitting infectious disease.

### WASTE MANAGEMENT

Please refer to local legal requirements.

**Date of issue:** 10. 2004.

### MANUFACTURER

#### P.Z. CORMAY

ul. Rapackiego 19, 20-150 Lublin, POLAND

P.O. Box 122, 20-954 Lublin 2

tel.: +48 (0) 81 749 44 00

fax: +48 (0) 81 749 44 34

<http://www.pzcormay.pl>

10/04/10/04