

CORMAY HbA_{1c} CALIBRATORS

CALIBRATORS FOR DETERMINATION OF HAEMOGLOBIN A_{1c} CONCENTRATION



Kit name	Kit size	Cat. No
CORMAY HbA _{1c} CALIBRATORS	1 x 8 ml + 5 x 2 ml	4-318

INTRODUCTION

Haemoglobin A_{1c} (HbA_{1c}) is a reaction product of glucose and the N-terminal group of beta chains.

The level of HbA_{1c} is proportional to the level of glucose in blood and has been widely accepted as an indicator of the mean daily blood glucose concentration over the preceding 6-8 weeks.

The measurement of HbA_{1c} is used in the long-term monitoring of diabetes mellitus.

PACKAGE

	Size
HbA _{1c} CALIBRATOR 1	1 x 8 ml
HbA _{1c} CALIBRATOR 2	1 x 2 ml
HbA _{1c} CALIBRATOR 3	1 x 2 ml
HbA _{1c} CALIBRATOR 4	1 x 2 ml
HbA _{1c} CALIBRATOR 5	1 x 2 ml
HbA _{1c} CALIBRATOR 6	1 x 2 ml

The calibrators are ready to use.

STABILITY AND STORAGE

Unopened calibrator is stable at 2-8°C until the expiration date printed on the vial. Opened calibrator store in original closed container. Avoid contamination.

WARNINGS AND NOTES

- Product for in vitro diagnostic use only.
- Do not pipette by mouth.
- Calibrators must be used only for the purpose intended by suitably qualified laboratory personnel, under appropriate laboratory conditions.
- Mix before use.

ASSIGNED VALUES

LEVEL	LOT. NO	HbA _{1c} g/dl	TOTAL HAEMOGLOBIN g/dl
1	504-01	0	18.0
2	504-01	0.47	-
3	504-01	0.78	-
4	504-01	0.99	-
5	504-01	1.54	-
6	504-01	2.06	-

WASTE MANAGEMENT

Please refer to local legal requirements.

Date of issue: 04. 2005.

MANUFACTURER

P.Z. CORMAY

ul. Rapackiego 19, 20-150 Lublin, POLAND
P.O. Box 122, 20-954 Lublin 2
tel.: +48 (0) 81 749 44 00
fax: +48 (0) 81 749 44 34
<http://www.pzcormay.pl>

04/05/04/05