

# CORMAY MYOGLOBIN CALIBRATORS

## CALIBRATORS FOR DETERMINATION OF MYOGLOBIN CONCENTRATION



<b>Kit name</b>	<b>Kit size</b>	<b>Cat. No</b>
CORMAY MYOGLOBIN CALIBRATORS	4 x 1 ml	4-279

### INTRODUCTION

Myoglobin (Mb) is a hemo-protein present in cardiac and skeletal muscle cells and is released into blood circulation when these cells are damaged. The determination of serum Mb level is useful in the diagnosis of myocardial infarction, muscular dystrophy, myositis and myopathy, and also for the assessment of treatment and disease prognosis.

CORMAY MYOGLOBIN CALIBRATORS kit is prepared by diluting purified human myoglobin with buffer solution containing 1 w/v% bovine serum albumin at various concentrations. Concentrations are shown on each vial and in this kit insert.

CORMAY MYOGLOBIN CALIBRATORS kit should be used for the preparation of the calibration curve for PRESTIGE 24i MYOGLOBIN, CORMAY MYOGLOBIN and CORMAY MYOGLOBIN 500 kits.

### PACKAGE

Cap colour	Mb concentration	Size
WHITE	100 ng/ml	1 x 1ml
BLUE	200 ng/ml	1 x 1ml
GREEN	400 ng/ml	1 x 1ml
RED	800 ng/ml	1 x 1ml

The calibrators are ready to use.

### STABILITY AND STORAGE

Unopened calibrator is stable at 2-10°C until the expiration date printed on the vial. Protect from light.

### WARNINGS AND NOTES

- Product for in vitro diagnostic use only.
- As the colours of caps vary according to myoglobin concentrations in the vial, care should be taken not to interchange the caps.
- The calibrators should not be mixed with ones in the other vial.
- The calibrator should be used according to this kit insert.
- The calibrator is based on human serum that has been tested for the HIV antibody, HBsAg and HCV and found to be non-reactive. However this material should be handled as though capable of transmitting infectious disease.

### WASTE MANAGEMENT

Please refer to local legal requirements.

**Date of issue:** 10. 2004.

### MANUFACTURER

**P.Z. CORMAY**  
ul. Rapackiego 19, 20-150 Lublin, POLAND  
P.O. Box 122, 20-954 Lublin 2  
tel.: +48 (0) 81 749 44 00  
fax: +48 (0) 81 749 44 34  
<http://www.pzcormay.pl>

10/04/10/04