

Tristel Duo

Tristel Duo is the first rapid action sporicidal cleaner for general hospital surfaces and non-invasive medical equipment. It comprises a highly effective bactericidal and virucidal wipe which is boosted by the application of the Tristel duo sporicidal foam, as and when required.

Features

- Combines a bactericidal and virucidal wipe with a sporicidal foam
- Highly effective cleaning system utilising Tristel's patented chlorine dioxide chemistry
- Effective against *clostridium difficile*; *bacillus subtilis*; *mycobacterium tuberculosis*; MRSA; *acinetobacter baumaii*; *norovirus*
- Instantaneous killing action
- Class IIa medical device



Tristel Duo

Applications

Suitable for use on all hospital surfaces and medical devices including:-

- Work surfaces
- Dressing trolleys
- Beds and mattress covers
- Instrument tables
- IV poles
- Monitor cases
- Control panels
- Pulse oximeters
- Key boards



Tristel Duo

Tristel Duo is a two-part system. Each Duo Pack contains 2 boxes of wipes and 2 foam pumps. When used alone the wipe is bactericidal, virucidal and fungicidal and should be used frequently throughout the day on all surfaces.

The Duo Foamer delivers chlorine dioxide directly to the surface which should be dispersed in conjunction with the wipe. The Duo foam should be used at the beginning and the end of each session. Additionally, the Duo foam should be used in the event of an outbreak or as frequently as directed by the Infection Control Team.

Microbiocidal activity

Organism	Contact time	Type of test	Data source	Test Result
<i>Bacillus subtilis</i>	30 seconds	Suspension	HIRL, Birmingham	> 6log reduction
<i>Mycobacterium tuberculosis</i>	30 seconds	Suspension	HIRL, Birmingham	> 6log reduction
<i>Bacillus subtilis</i>	30 seconds	Surface	Dr David Coates	> 6log reduction
<i>MRSA</i>	30 seconds	Suspension	Glasgow Caledonian University	> 6log reduction
<i>Norovirus</i>	30 seconds	Suspension	Glasgow Caledonian University	> 6log reduction
<i>SARS</i>	30 seconds	Suspension	Glasgow Caledonian University	> 6log reduction

